
 Bellman & Symfon®


**BE1433**  
**Mobile phone transceiver**  
**Bellman & Symfon Group AB**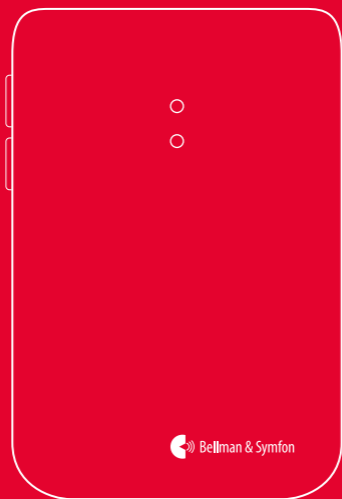

# BE1433

## Mobile phone transceiver

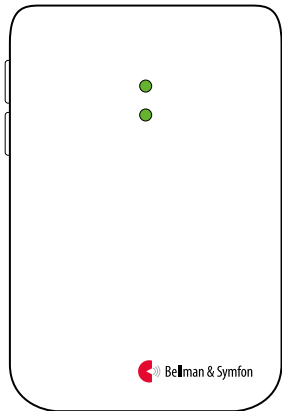4	Bedienungsanleitung	DE
18	Brugervejledning	DK
32	User manual	EN
46	Manual de usuario	ES
60	Käyttöopas	FI
74	Mode d'emploi	FR
88	Gebruiksaanwijzing	NL
102	Brukermanual	NO
116	Bruksanvisning	SE

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DK

## Read this first

This booklet guides you on how to use and maintain your new Mobile phone transceiver. Ensure you read it carefully, including the **Warnings** section. This will help you get the most benefit from your product. If you have additional questions, contact your hearing care professional.

### **Intended purpose**

The intended purpose of the Visit alerting system is to alert deaf and hard of hearing people of important signals in their home.

### **Intended user group**

The intended user group consists of people of all ages with mild to severe hearing loss or deafness that need audio, visual or sensory amplification.

EN

### **Intended user**

The intended user is a person with mild to severe hearing loss or deafness.

### **Principle of operation**

The Visit alerting system consists of a set of wirelessly connected transmitters and receivers that are located across the home. When a transmitter detects an activity, it signals the Visit receiver that alerts the user with sound, flashes or vibrations. In order to use the system, you need at least one transmitter and one receiver.


# Let's get started!

## What you'll need

- A mobile phone with iOS 11 / Android 6 or later.

## What's in the box

- 1 Mobile phone transceiver
- 2 Power cable
- 3 Telephone cord
- 4 Screw and plug
- 5 Telephone splitter
- 6 Power adapter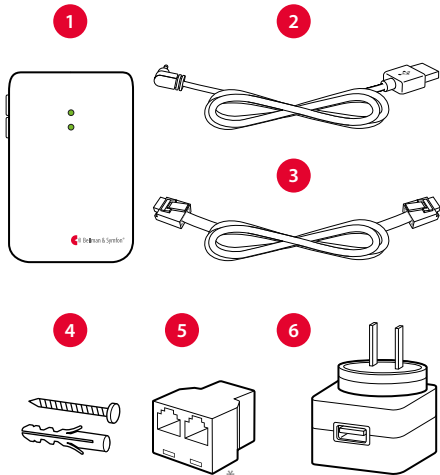


\* The appearance of your telephone splitter may differ.

# Installing the app

## 1 Turn on Bluetooth®

Swipe down from the top of the screen and tap the Bluetooth icon.

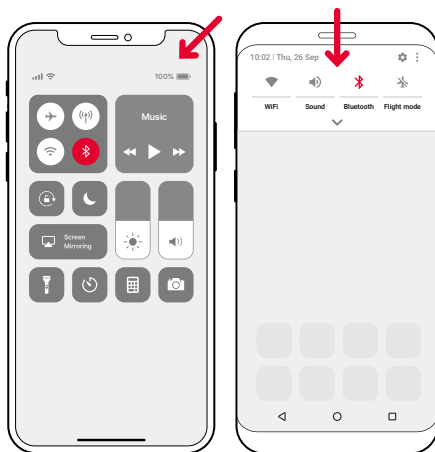EN

## 2 Install the Visit app

Download the **Bellman Visit** app from the App Store® or Google Play™ and follow the setup instructions.

# Installing the transceiver

## 1 Mount it

Remove the protective film from the Velcro on the back of the transmitter and mount it on the wall. Alternatively use the supplied screw and plug.

## 2 Turn it on

Pull the battery tab to start the transmitter. The indicators light up in green to show that it is on.

## 3 Connect it



Connect the USB cable to the power adapter. Then plug the adapter into a wall outlet. If you have a landline telephone, connect it to the telephone jack using the supplied splitter.

## 4 Test the mobile phone

Have someone call your mobile. The yellow LED on the Visit receiver blinks and it starts to flash, sound or vibrate (depending on the receiver).

## 5 Test the landline phone

Call your landline phone. The yellow LED on the Visit receiver lights up and it starts to flash, sound or vibrate (depending on the receiver).

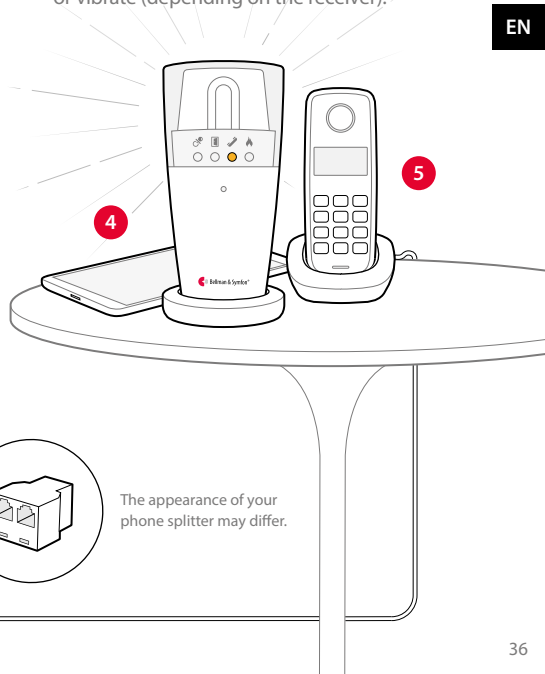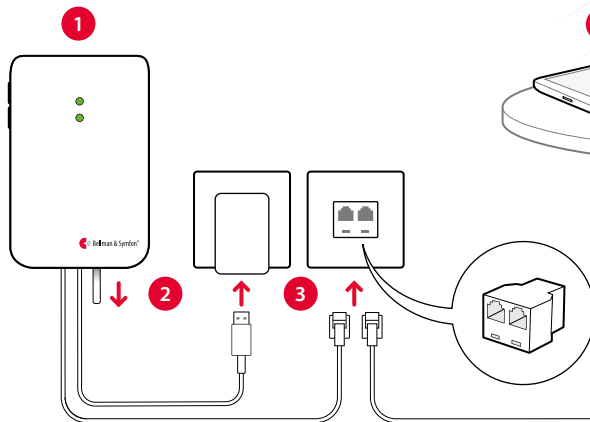The appearance of your phone splitter may differ.

# Using the app

Once you have installed the Visit app, you will start to receive Visit notifications on your mobile device. In addition, your Visit receiver will alert for mobile calls and messages. Controlling your system from the app is easy. Here's what you can do:

## Manage notifications

Enable, disable or delete a notification with an easy tap.

## Control your devices

Get an overview of your connected devices and tap + to add a new.


## View history

See your latest notifications, sorted by date and time.

## Learn more

Tap **User guide** in the menu to learn more about the features.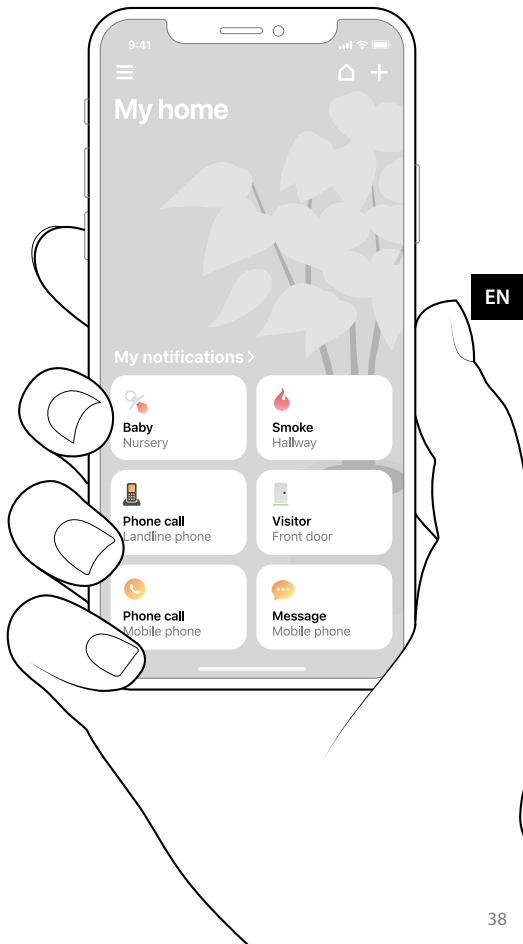

# General warnings

This section contains important information about safety, handling and operating conditions. Keep this booklet for future use. If you are just installing the device, this booklet must be given to the householder.

## Hazard warnings

- Failure to follow these safety instructions could result in fire, electric shock, or other injury or damage to the device or other property.
- Make sure to keep this device out of reach of children under 3 years.
- Note that alarms and notifications can be missed if batteries run out.
- This product uses Alkaline batteries as power backup. Do not expose the batteries to fire or to direct sunlight. Only use 1.5 V AA alkaline batteries.
- This device is designed for indoor use only. Do not expose the device to moisture.
- Protect the device from shocks during storage and transport.
- This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the user guide, may cause harmful interference to radio communication.
- Do not use or store this device near any heat sources such as naked flames, radiators, ovens or other devices that produce heat.
- Do not dismantle the device; there is a risk of electric shock. Tampering with or dismantling the device will void warranty.
- Protect cables from any potential source of damage.

- Do not make any changes or modifications to this device. Use only power adapters and accessories that are specified in this booklet.
- Batteries are toxic. Do not swallow them! Keep out of reach of children and pets. If they are swallowed, consult your physician immediately!

## Information on product safety

- Failure to follow these instructions could result in damage to the device and void the warranty.
- Do not expose batteries to fire or to direct sunlight.
- Do not drop your device. Dropping onto a hard surface can damage it.
- Do not use the device in areas where electronic equipment is prohibited.
- The device may only be repaired by an authorized service center.
- If a serious incident occurs in relation to this device, contact the manufacturer and relevant authority.
- If you encounter other problems with your device, contact the point of purchase, your local Bellman & Symfon office or the manufacturer. Visit **bellman.com** for contact info.
- This device should not be used on aircrafts unless specifically permitted by flight personnel.

EN

## Configuration options

This transceiver can be configured with the following Visit receivers and accessories:

### Compatible Visit receivers

- BE1450 Visit portable receiver
- BE1441 Visit flash receiver
- BE1442 Visit flash receiver
- BE1580 Visit alarm clock receiver

- BE1470 Visit pager receiver
- BE1560 Visit wrist receiver

### Available accessories

- BE9026 Contact mat
- BE9023 Magnetic switch
- BE9253 External trigger cable

For detailed product information, see the corresponding instructions for use.

## Operating conditions

Operate the device in a dry environment within the temperature and humidity limits stated in this booklet. If the product gets wet or is exposed to moisture, it should no longer be regarded as reliable and should therefore be replaced.

## Cleaning

Disconnect all cables before you clean your device. Use a soft, lint-free cloth. Avoid getting moisture in openings. Do not use household cleaners, aerosol sprays, solvents, alcohol, ammonia, abrasives or compressed air.

## Service and support

If the product appears to be damaged or doesn't function properly, follow the instructions in this booklet. If the product still doesn't function as intended, contact your local hearing care professional for information on service and warranty.

## Warranty conditions

Bellman & Symfon guarantees this product (excluding the battery) for two (2) years from date of purchase against any defects that are due to faulty materials or workmanship. This guarantee only applies to normal conditions of use and

service, and does not include damage resulting from accident, neglect, misuse, unauthorized dismantling, or contamination howsoever caused.

This guarantee excludes incidental and consequential damage. Further the warranty does not cover Acts of God, such as fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights and you may also have other rights that vary from state to state. Some states or jurisdictions do not allow the limitation or exclusion of incidental or consequential damages, or limitations on how long an implied warranty last so the above limitation may not apply to you.

This guarantee is in addition to your statutory rights as a consumer. The above warranty may not be altered except in writing signed by both parties hereto.

## ISO Certification of legal manufacturer

Bellman & Symfon is certified in accordance with SS-EN ISO 9001 and SS-EN ISO 13485.

SS-EN ISO 9001 Certification Number: CN19/42071  
SS-EN ISO 13485 Certification Number: CN19/42070

### Certification Body


SGS United Kingdom Ltd  
Inward Way  
Rossmore Business Park  
Ellesmere Port, Cheshire, CH65 3EN  
United Kingdom

The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by Bellman & Symfon is under license.

## Regulatory symbols
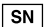
 This symbol confirms that the product meets the Medical Device Regulation EU 2017/745.

 This symbol indicates the manufacturer's serial number so that a specific medical device can be identified. It's available on the product and gift box.

 This symbol indicates the manufacturer's catalogue number so that the medical device can be identified. It's available on the product and gift box.

 This symbol indicates the medical device manufacturer, as defined in EU Directives 90/385/EEC, 93/42/EEC and 98/79/EC.

 This symbol indicates that the user should consult the instructions for use.

 This symbol indicates that it is important for the user to pay attention to the relevant warning notices in the instructions for use.

 This symbol indicates important information for handling and product safety.

 Temperature during transport and storage:  
-10° to 50° C, 14° to 122° F  
Temperature during operation: 0° to 35° C, 32° to 95° F

 Humidity during transportation and storage: <90%, non-condensing. Humidity during operation: 15% to 90%, non-condensing.

 Atmospheric pressure during operation, transportation and storage: 700hpa to 1060hpa.

**Operating conditions**  
This device is designed such that it functions without problems or restrictions if used as intended, unless otherwise noted in the instructions for use.

 With this CE symbol, Bellman & Symfon confirms that the product meets EU standards for health, safety, and environmental protection as well as the Radio Equipment Directive 2014/53/EU.

 This symbol indicates that the product shall not be treated as household waste. Please hand over your old or unused product to the applicable collection point for the recycling of electrical and electronic equipment.

## Compliance information

Hereby Bellman & Symfon declares that, in Europe, this product is in compliance with the essential requirements of the Medical Device Regulation EU 2017/745 as well as the directives and regulations listed below. The full text of the declaration of conformity can be obtained from Bellman & Symfon or your local Bellman & Symfon representative. Visit [bellman.com](http://bellman.com) for contact information.

- Radio Equipment Directive (RED)
- Medical Device Regulation (MDR)
- Waste Electrical & Electronic Equipment (WEEE)
- RoHS Directive
- REACH Regulation
- EC Battery Directive

## Technical specifications

**Dimensions** 100 × 65 × 27 mm, 4.0 × 2.6 × 1.1" EN

**Weight** 198 g, 6.98 oz. including batteries, telephone cable and power cable

**Mains power** 5 V DC / 1000 mA

**Backup batteries** 2 × 1.5 V AA Alkaline

**Connectivity** RJ11 input to analogue telephone network, 3.5 mm ext. trig input, Power supply jack, Bluetooth Low Energy, BLE

**Frequency** ISM frequency: 868.30 MHz  
Transmission power: 4mw (max.)  
Bluetooth freq: 2400 – 2483.5 MHz  
Transmission power: 4mw (max.)

**Coverage** ISM: Up to 250 m, depending on the characteristics of the building.  
Bluetooth: Up to 200 m

**Compatibility** iOS11 and later / Android 6 and later  
iPhone 6S and later / Modern Android devices

**In the box**

- BE1433 Mobile phone transceiver (including batteries)
- Screw and plug
- Power adapter and power cable
- Telephone splitter
- Telephone cord

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All rights reserved.
TM & © 2022 Bellman & Symfon AB.